



The following cleaning methods are recommended for the KardiaMobile and KardiaMobile 6L products:

1. Clean the electrodes by wiping with a soft cloth dampened with water or one of the following approved cleaners:
 - a. Soap and water, or
 - b. Bleach solution as recommended by the CDC:
<https://www.cdc.gov/coronavirus/2019-ncov/prevent-getting-sick/disinfecting-your-home.html>
2. To clean, spray the cleaner on a soft cloth and thoroughly wipe the device.
3. Ensure the device is sufficiently dried.

Precautions:

1. Do not use disinfecting wipes or alcohol based products as these products could adversely affect the product performance.
2. Do not immerse the device or expose the device to excessive liquid.